



INSTRUCTION MANUAL

Product: Positioning Aids
MDR-003B | Version 00
08.03.2021

Application:

The positioning aids are intended for positioning the patient's head, trunk or outer extremities in medically desirable positions. The positioning aid and cover may only be used by instructed specialist personnel in clinics, surgeries or other care facilities. The case in which the positioning aid is delivered is used exclusively for storing the individual positioning aids and does not serve any medical purpose.

Patient positioning:

The positioning aids may only be used by trained specialist personnel. The position and use of the positioning aid is specified and selected by the trained specialist personnel.

Storage and handling:

The product must be stored at room temperature, dry and protected from light.

Instructions for use:

- Monitoring of the patient during treatment is required to prevent incorrect positioning.
- The trained specialist personnel decides which positioning aid is to be used for the respective required application.
- The trained specialist personnel is responsible for ensuring that the intended positioning aid is used for the corresponding body part.

Warning:

- Do not rinse, soak in liquids, disinfect or sterilize the product. Liquids can penetrate the positioning aid and damage it.
- Use product only at room temperature.
- Before using the positioning aids, a visual inspection must be carried out. If there is visible damage to the surfaces, e. g. cracks, the product must not be used. This also applies if the product does not return to its original shape after use or shows dents or similar.
- The positioning aid is intended exclusively for use with intact skin. If the skin is injured, the wound must be properly covered before positioning.
- Regional regulations must be observed for disposal.
- The positioning aids or their covers are not connected to the patient. Parts of the patient's body are simply placed on it, without additional fixation.

Care instructions

- To clean the synthetic leather cover, we recommend wiping it with warm water and adding a liquid mild de-tergent. It is possible to wash the synthetic leather cover at 30°C and dry it in the dryer with reduced thermal load up to 60°C. Dry cleaning is not possible. Only the textile back of the cover may be ironed at hand temperature.
- Positioning aids with Skinfoam coating or thermoformed positioning aids can be disinfected or cleaned. Suitable substances include ethanol (undiluted), surgical spirit (undiluted), as well as common household wash-up liquids and washing-active substances (10 % by volume). The manufacturer's instructions must always be followed.

The instructions given are not guarantees and do not release the user from his duty of care in handling the materials and cleaning agents.



Manufacturer:
Polyform Kunststofftechnik
GmbH & Co. Betriebs KG
Braasstraße 15
31737 Rinteln

